


# Guedel Airway

*single use*

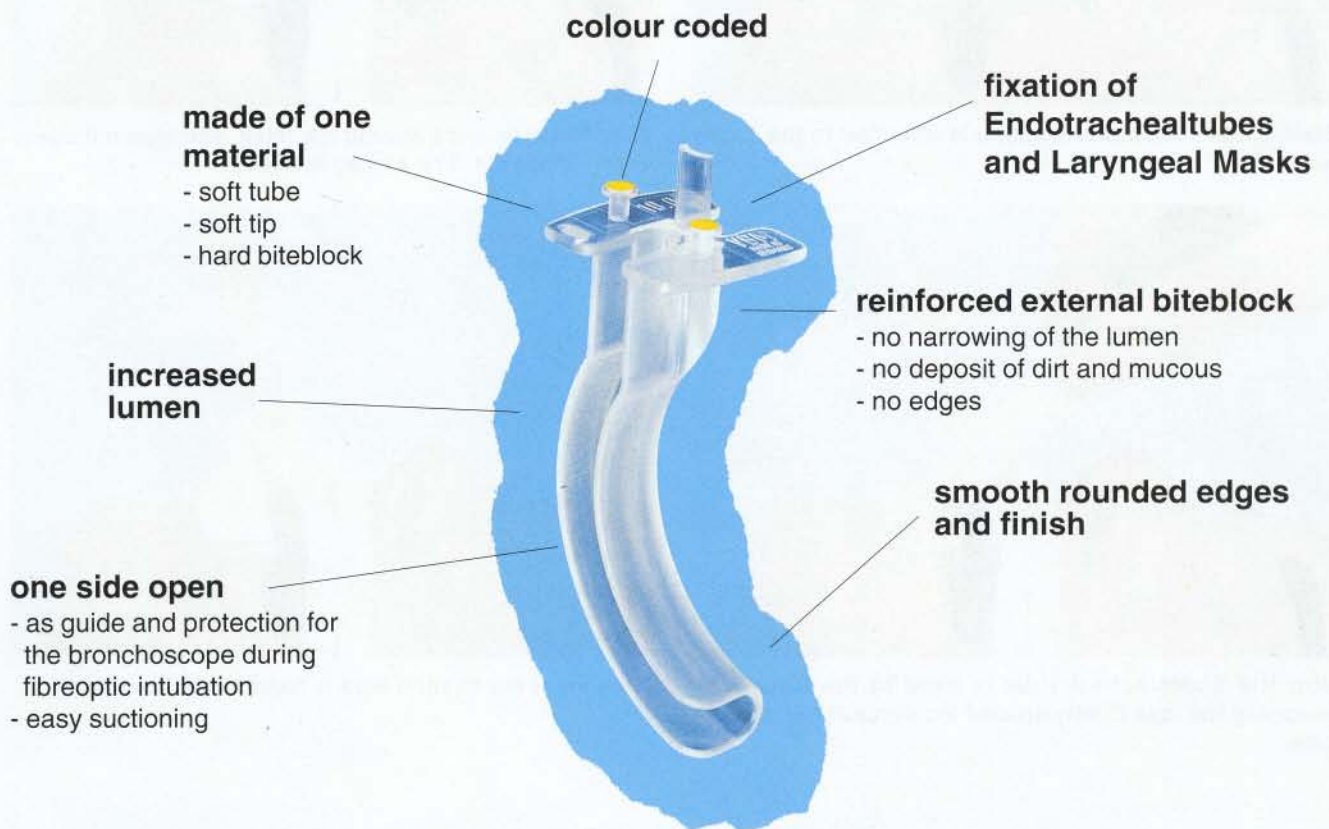

## GUEDEL AIRWAY, for single use

material: EVA


single steril packed

size	length (cm)	colour code	Ref. No.	unit/box
000	3.5	light blue	40-20-007-1	50 pcs.
00	5.0	marine	40-20-008-1	40 pcs.
0	6.0	black	40-20-009-1	30 pcs.
1	7.0	white	40-20-010-1	20 pcs.
2	8.5	green	40-20-020-1	30 pcs.
3	9.0	yellow	40-20-030-1	30 pcs.
4	10.0	red	40-20-040-1	30 pcs.
5	11.0	violet	40-20-050-1	30 pcs.


# "Guedelfix" *single use*

## For fixing Endotracheal Tubes and Laryngeal Masks

Instruction for use:

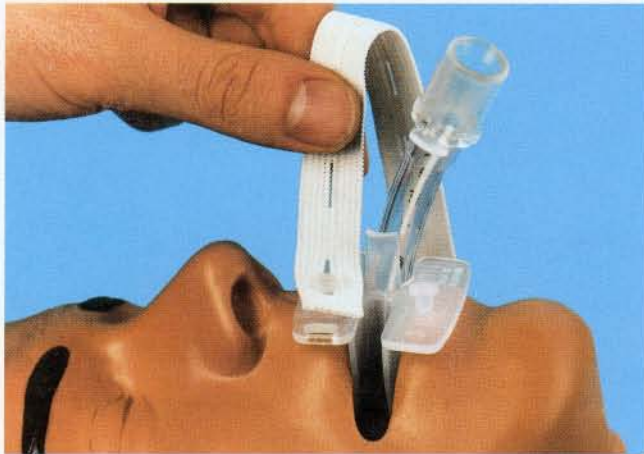


Elastic tape or Foam Necktape is attached to the upper pin.

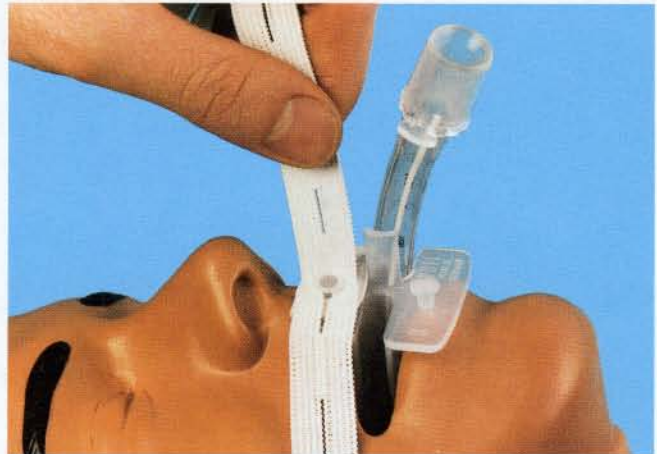

Wrap the tape once around the neck and attach it again to the upper pin. The Airway is fixed.



Now the Endotracheal Tube is fixed to the Airway by wrapping the tape tightly around the support bar and the tube.

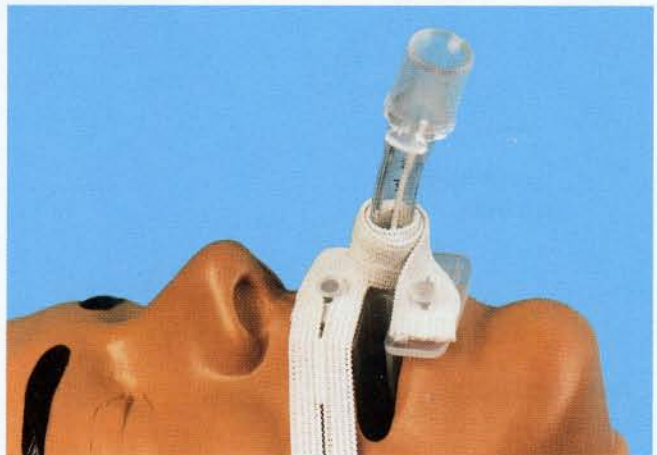

The end of the fixation tape is fixed to the lower pin.

### **GUEDELFIX, single use**

material: EVA

single sterile packed

size	length (cm)	colour code	Airway with foam necktape	Airway with elastic necktape	unit/box
000	3.5	light blue	--	40-26-007-1	50 pcs.
00	5.0	marine	--	40-26-008-1	40 pcs.
0	6.0	black	--	40-26-009-1	30 pcs.
1	7.0	white	40-25-010-1	40-26-010-1	20 pcs.
2	8.5	green	40-25-020-1	40-26-020-1	30 pcs.
3	9.0	yellow	40-25-030-1	40-26-030-1	30 pcs.
4	10.0	red	40-25-040-1	40-26-040-1	30 pcs.
5	11.0	violet	40-25-050-1	40-26-050-1	30 pcs.

### **Fixation tapes**

#### **Foam necktape**

length	Ref. No.	unit/box
74 cm	40-25-000	5 pcs.

#### **Elastic necktape**

length	Ref. No.	unit/box
40 cm	40-06-040	10 pcs.
55 cm	40-06-055	10 pcs.
75 cm	40-06-075	10 pcs.